

MB120 FaceShield


1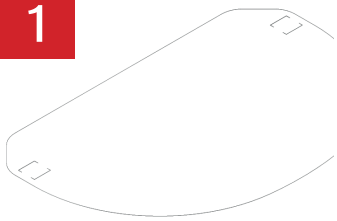


2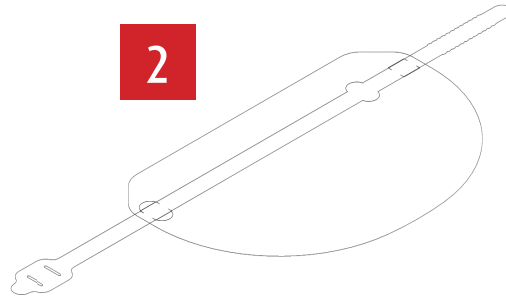


3

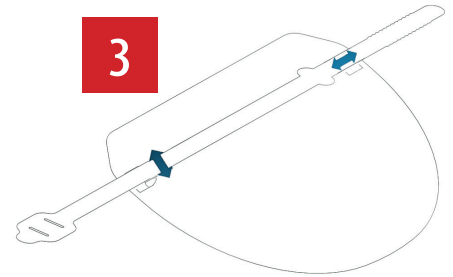
4

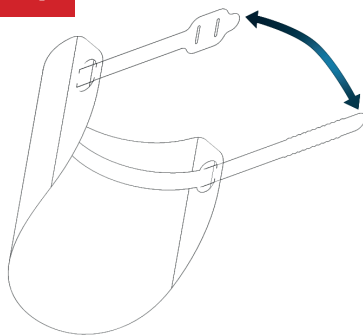5

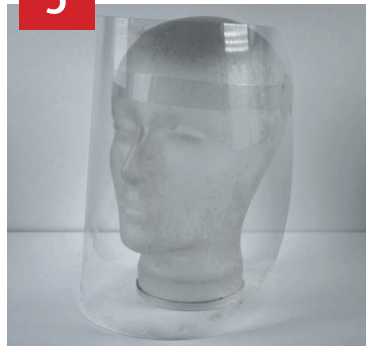6

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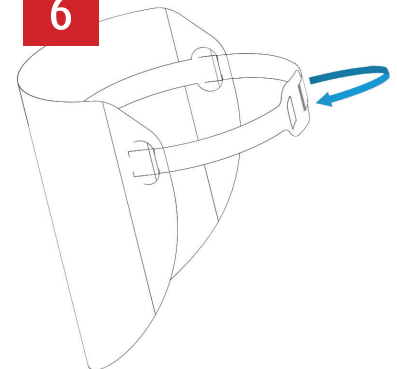Assembly Instructions: ① Place the shield on a smooth surface. The surface should be such that the shield surface cannot be scratched. It is best to place a soft cloth on the underlay. ② Place the headband on the shield. Position the crescent moon tabs on the headband (top and bottom) over the holes on the right and left at the upper part of the shield. ③ Insert the upper crescent flap into the left punched hole on the shield, then insert the lower crescent flap into the lower punched hole. Then follow the same procedure on the right side of the shield. Insert the upper crescent flap into the left punched hole on the shield, then insert the lower crescent flap into the lower punched hole. Then follow the same procedure on the right side of the shield. ④ Bring the two ends of the headgear together so that the push-in tab with the wave-shaped catches is inserted into the outer closing gap from the inside out. ⑤ Now put on the FaceShield so that it lies comfortably on the forehead at the front over the eyebrows and at the same height at the back of the head and ensures the necessary hold. ⑥ Now insert the insertion tab into the locking gap from the outside in.

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Watch the video
for assembly here!

Assembly instructions in
your language as PDF here!